QUICK REFERENCE GUIDE


For *in vitro* diagnostic use only.
For use with fresh, capillary or fresh, venous whole blood samples taken from a dog or cat.
NOT FOR HUMAN USE.

Before You Test

- ✓ Seek the guidance of a veterinarian before monitoring blood glucose.
- ✓ ALWAYS read the Owner's Booklet before testing for the first time and before using this Quick Reference Guide.

Blood Glucose Testing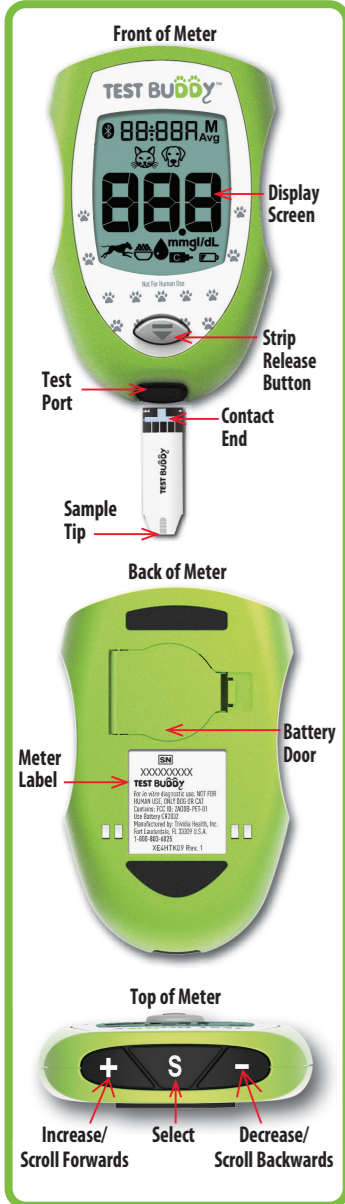


Step 1
Gather your materials and prepare your lancing device. Set the animal type (🐕 dog or 🐈 cat) when you use the meter for the first time.

Step 2
Wash your hands and the test site (e.g. ear or paw pad) with warm water and soap to ensure accurate results. Rinse and dry thoroughly.

Step 3
Remove test strip from vial and close vial immediately. Insert test strip contact end (black blocks facing up) into the meter test port. The meter turns on. The display shows the time, animal symbol, and blinking drop. Make sure that the animal symbol matches the animal you are testing.

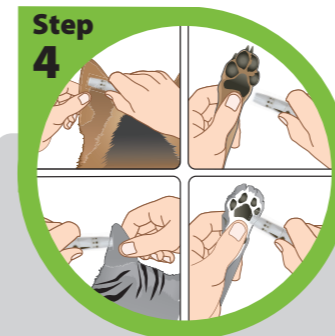Step 4
Lance the test site (ear or paw pad). Allow blood drop to form. Refer to *Lancing Device Instructions for Use* for detailed instructions.

Step 5
With test strip still in meter, touch the test strip sample tip to the top of the blood drop. Hold the test strip to the blood drop until the meter beeps and dashes move across the display.

Step 6
The glucose result is displayed with alternating date and time. Mark the result with an Event Tag or record in the logbook.

Step 7
Hold the meter with the test strip pointing down. Press the strip release button to discard the test strip into a suitable waste container. Dispose used test strips and lancets according to local regulations.


WARNING: After opening the test strip carton, check the test strip vial for missing, damaged, or broken parts. Check that the cap is securely closed. If the vial is damaged or the cap is not closed, do not use the test strips for testing as the test strips may give inaccurate results. Contact Customer Care at 1-800-803-6025, Mon. - Fri., 8AM-8PM EST for replacement.

Includes Four Event Tags for Greater Insight

The Test Buddy™ system features advanced event tags that combine pets' activities with their results.

Step 3 Tips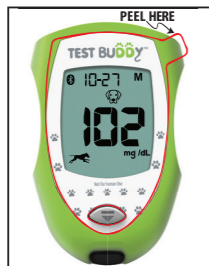

Important! Remove the protective plastic overlay on the display screen.

Remove one test strip from vial and immediately press down vial cap firmly to close. See *Additional Information* for storage conditions.

Insert test strip contact end (black blocks facing up) into the test port.

DO NOT place test strip in backwards.


Step 5 Tips

With strip in meter, touch tip of test strip to the top of the blood drop.

DO NOT place blood onto test strip before inserting test strip into meter test port.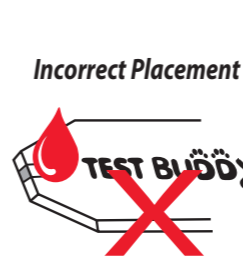

DO NOT place blood drop on top of test strip or any other location except the sample tip of the test strip.

Additional Information

- ✓ ALWAYS store the test strips in the original vial with the cap tightly closed.
- ✗ NEVER transfer test strips from one vial to another. Do not expose to heat, moisture, or humidity.
- ✗ NEVER compare the Test Buddy™ meter to other meters.
- ✗ NEVER immerse meter or test strips in liquids or place any liquids onto meter or into the test port.
- ✓ Store the Test Buddy™ System at room temperature 36°F-86°F. **DO NOT REFRIGERATE OR FREEZE.**
- ✓ Wash hands thoroughly with soap and warm water before and after handling the meter, lancing device, lancets, or test strips as contact with blood presents an infection risk.
- ✓ Keep the meter and the lancing device clean and disinfected. For cleaning and disinfecting instructions see *Meter Cleaning and Disinfecting* and *Lancing Device Cleaning and Disinfecting* in the *Owner's Booklet*.

Customer Consultation
Monday - Friday, 8AM - 8PM EST
1-800-803-6025
www.testbuddymeter.com

Manufactured by:
TRIVIDIA
HEALTH™
FORT LAUDERDALE, FL 33309 U.S.A.

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